

HealthWizard Update Installation Instructions

Mar 3, 2018

1. Click the link on our website page to download the HealthWizard installation file.

The file will be located in your computer's Downloads folder. The file name will be **HealthWizardxxx_nnnn.exe** where xxx is a version number and nnnn is an ID number.

2. Move the HealthWizard installation file to a safe, long term storage location.

You may have a computer failure in the future and need to install HealthWizard again.

3. Copy the installation file to the computer that is running the old HealthWizard software.

It is common to copy the file to the Desktop folder.

4. Double click the HealthWizard installation file to start the installation process.

You may see a security message warning the software publisher cannot be verified. Ignore the message and click the **Run** button.

5. Follow the on-screen instructions.

You may see (two times) a message saying 'Do you want to allow the following program from an unknown publisher to make changes to this computer?' Ignore these messages and click the **Yes** button.

6. When the installation program is complete, you are done.

Important: if you have two or more computers running HealthWizard that share the same database on a network, be sure to update HealthWizard on all computers.