

# HealthWizard - Version 5.3.8

April 1, 2012

## Fitness Profile

New: Export client fitness test scores to Excel file  
A new feature found in Reports > Group Test Scores > Fitness Profile. You can export data for all clients in your database or select a subset of clients based on parameters like, client group type/name, age range, gender, date range of test session, test format or single fitness test. Report is written to an Excel file.

New: Calibrate MicroFit body weight scale using calibration number  
The MicroFit body weight scale can now be calibrated by entering the Scale Calibration Number found on the bottom of the body weight scale.

After you install HealthWizard, this will be the default calibration mode. If your MicroFit body weight scale is old and does not have a Scale Calibration Number on the bottom, you can switch back to the old calibration method of using the body weight from a person. To do this the user must go to Utilities > Configure System and select the Fitness Profile tab. Then check 'Use body weight to calibrate weight scale'.

If you have an old MicroFit body weight scale and would like a Scale Calibration Number, call MicroFit at 559-475-7007 and select Customer Service. You may also email [support@microfit.com](mailto:support@microfit.com) for information. service.

Enhance: Fitness Profile Group Reports: new Total Fitness Score calculation  
In prior versions, the Total Fitness Score for the Fitness Profile Group Report, Group Comparison Report and Trend Report was calculated as the average of all the individual client Total Fitness scores. In this version the Total Fitness Score for these group reports is calculated from the single set of averaged test scores.

## General

New (5.3.5): Compatible with Windows 7 32-bit and 64-bit  
HealthWizard is compatible with Windows Vista and Windows 7 for both 32-bit and 64-bit versions. It is still compatible with Windows XP.

Bug Fix: Custom report header not showing in some reports  
When 'Use custom report header' is selected, several reports were not displaying the custom report header.