



WELLNESS ASSESSMENT SOFTWARE & TESTING EQUIPMENT

## HealthWizard Version 5.4.5

March 28, 2014

### Major New Features

1. Send a copy of an individual or group report by email (requires MicroFit Manager).

### MicroFit Manager

New Feature: Email HealthWizard Reports.

You can now email any HealthWizard report. In prior versions of HealthWizard you had the report printing options of *Display/Print*, *Print Only*, and *Print to File*. The *Print Only* option has been replaced with *Email*.

#### Requirements for Email Feature

1. The MicroFit Manager software component must be licensed for use.
2. The computer where HealthWizard is installed must have Internet access.
3. You must have an active email account from Google, Yahoo, or other email hosting service to use as the "From Email".
4. Most reports are sent as a PDF file attachment. The recipient must have Adobe Acrobat Reader installed on their PC, tablet or phone to view these reports. A free download is available from Adobe.
5. The Fitness Profile Export report is sent as an Excel file attachment. The recipient must have Microsoft Excel installed on their PC, tablet or phone to view this report.

Update: Settings Menu - move Custom Report Header option to Settings menu.

The *Custom Report Header* menu option has been moved from the Manager menu to the new Settings menu.

Update: Manager Menu – hide menu when MicroFit Manager not installed.

The Manager menu name and the *Custom Report Header* and *Email* menu options (under Settings menu) are now hidden when the MicroFit Manager software module is not licensed.

## **HealthWizard**

- Update: Utilities menu has been renamed to Settings menu.  
The HealthWizard Utilities menu has been renamed to Settings. Edit *Organization Name*, use *Password Security* and *Configure Test Devices* have been moved from Utilities > Configure System to the Settings menu. *Calibrate Test Devices* has been moved from Utilities menu to the Settings menu.
- New Feature: Log In password input form appears immediately.  
If HealthWizard is set to use security password, you will now see the Log In password input form when the program is started. In the past you had to click File > Log In to show the Log In password input form.
- Update: Installation – cannot select install to folder  
The HealthWizard installation program no longer has the option to select the “install to” folder. HealthWizard is now always installed to the computer system default program folder.

## **Fitness Profile**

- Bug Fix: Aerobic Fitness Test – unusual behavior during testing.  
Some users have reported the following unusual behaviors during an aerobic fitness test: no cadence sound during Step Test, program freezing up at the start of an aerobic fitness test and flashing buttons that do not respond to a mouse click. The code has been updated to eliminate these problems.
- Bug Fix: Add Fitness Test - No button is now the default for time test.  
When you add a new fitness test, a message box appears asking if the scores for the new test will be entered in a min:sec format. The default button is now the *No* button.
- Bug Fix: Edit Client Test Scores – error during editing.  
When editing the Fitness Profile test scores for a client, if the cursor was in the last (bottom) score input box and the Enter key was pressed, an error message appeared.
- Bug Fix: Client Reports – unusual behavior preparing to print the report.  
When you select Reports > Client Test Scores > Fitness Profile and then select a report to print for a client who has no test data, you first see the ‘No test data’ message. Then the report definition form would appear with no client selected (a confusing situation). Now the report definition form does not appear under this condition.